BED HANGER ARTICLE NO. RC876

Thank you for choosing this product. If you observe a few important things, the product will be helpful to you for many years and make your everyday life easier. You should therefore read these operating instructions carefully. Always observe the following safety instructions.

SAFETY INSTRUCTIONS:

Always ensure that you have a secure footing. Only use the lifting pole on a flat, firm surface. Avoid high lateral loads. Never use it for transporting it from bed to armchair or wheelchair. When using it on bed or on adjustable armchairs, ensure that there is sufficient distance from moving parts. Before each use, make sure that the seams, the material of the loop and the fastening elements of the triangle handle are free of damage and securely fastened. For your own safety, change these parts every five years. Do not overload the lifting pole. The maximum load capacity is 90 kg. The lifting aid must not be rocked.

SCOPE OF APPLICATION:

The lifting pole is only suitable for indoor use. In conjunction with most standard beds and due to the low foot height, it can also be used as a stand-up aid for seating furniture. Do not place the lifting pole to the side of the object, but always behind the object at the head or back end. No additional fixings on the wall or on the bed are required. The handle height is just as adjustable as the handle position above the bed or armchair. The lifting pole can be easily dismantled and therefore easily transported and stored. The maximum load capacity is 90 kg. This is suitable for a maximum body weight of 130 kg when used on the bed. If you do not use the product for a long

SeniorSam®

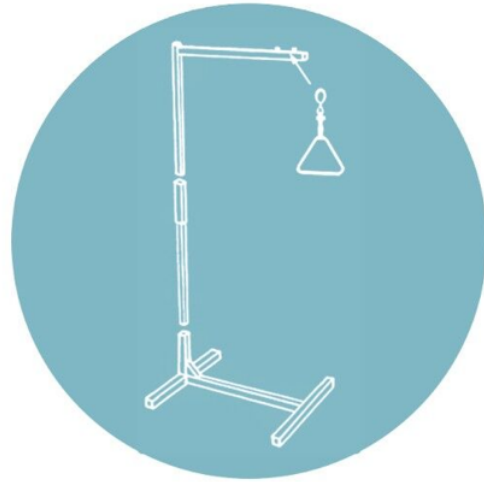
SeniorSam A/S
Sundbyvej 19, 7752 Snedsted Danmark

period of time, store it in a dry place and protected from dust and direct sunlight.

STRUCTURE AND APPLICATION:

Make sure that the packaging is intact during delivery. Any damage must be reported immediately to the delivery personnel. Later complaints can no longer be accepted. Remove all parts from the packaging. All parts of the packaging are recyclable and can be disposed of with the household waste according to local regulations.

Then attach the triangle handle and adjust the height of the triangle handle to the desired length. The product is now ready for use. Position the lifting pole according to the desired application behind the bed or armchair. However, always make sure that the extension arm with the triangle handle is centered above the patient and that the patient does not put any weight on the lifting pole when using it. The triangle handle can be adjusted both in height and in position. To do this, the loop has to be lengthened or too short.

CLEANING AND DISPOSAL:

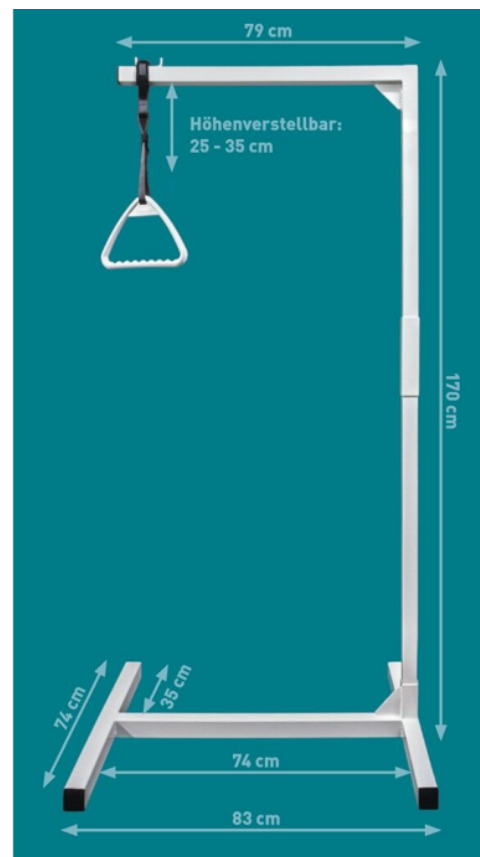
You can wipe all metal parts and the triangle handle with a mild, non-abrasive household cleaner and a soft cloth. For disinfection, we recommend a commercially available spray disinfection. be sent for recycling.

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