User Manual UK

Product: Aluminum Walker

Item No.: RC16

First of all, thank you very much for choosing SeniorSam products. The design and manufacture of this product meet the relevant product standards. Please read the instructions carefully before using this product and keep it for future reference.

Before using this product, please make sure you fully understand the following warnings:

1. People with reduced mobility, seniors, and those who cannot help themselves must be looked after by others.
 2. The function of this product is to assist walking. Never support the weight of the whole body on the product.
Do not use the walker with only one foot on the ground.
 3. Please pay attention to whether the ground is flat or slippery during use to ensure safety in use.
 4. Load: 136kg.
 5. After use, wipe the surface of the product with a dry towel and place it in a ventilated place;
Do not use steel brushes and abrasive cleaners to avoid damaging the product surface.
 6. Make sure all the main body and accessories are intact and firm, if the product foot pad is found to be worn and cracked, please replace immediately.
 7. Improper use can easily cause deformation and fracture of this product.
- We are not responsible for your physical injury.

II. Using Instruction!

Turn on the walker:

Spread the left and right sides of the walker outward until the button are fully ejected.
Please make sure the button has popped up before use.

2. Collapse walker:

After pressing down the tube sleeve on the walker, close the left and right grips inward

3. Height adjustment:

It can be adjusted to the height suitable for users according to different heights.

Press down the marble on the foot tube and adjust it to a suitable adjustment hole to make the marble **pop out**.

Note:

Please be sure to have four feet adjusted in the same level to avoid danger.

How to use:

When in use, the four feet of the walker should be grounded first, and the user should hold the grips on both sides.

Lift the walker forward and take a small step down, the user is moving forward.

III. Repair and maintenance:

1. Regularly check whether the screws are loose and whether the rubber feet are firm and stable. Replace them if they are worn.
2. Always clean, disinfect regularly, and place in a dry and ventilated environment.

VII. Precautions, warnings, and instructions:

1. Objects used by the product

(1) It is used by patients with reduced mobility, seniors, and those who cannot help themselves must be looked after by others.

(2) Please do not allow infants and young children and people who cannot understand the operation to use this product alone.

2. Potential safety hazards and use restrictions

(1) Before using the device, please read the instruction manual carefully. If this equipment is used in a manner not specified in the instructions, the protection provided by the equipment may be impaired.

(2) When starting to use the device, the user should practice in a flat and wide place, in order to be proficient in the various characteristics of crutch, in order to avoid the aging of the rubber feet during the use process, which may cause the user to be injured.

(3) When the walker is not in use or needs to be moved, the walker should be folded or placed in a corner to avoid occupying extra space and preventing others from walking.

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