Teeth Whitening Kit

NL• Tandbleekset FR• Kit de Blanchiment Dentaire DE• Zahnaufhellungs Set

Manual

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EN • How to use:

- Brush your teeth with toothpaste and rinse with water.
- Take the batteries out of the LED light, remove the plastic insulating strip and replace the batteries.
- Clean the mouth tray with lukewarm water, dry and attach the mouth tray to the LED light.
- Apply 1/3 (0.5 ml top + 0.5 ml bottom) syringe of whitening gel into each side of the mouth tray, apply just enough to coat the frontal surfaces of your teeth to be whitened.
- Place the mouth tray over your teeth and wipe off any gel that's in contact with your gums.
- Press the start button of the LED light and close your mouth during 15 minutes use.
- Rinse your teeth after 15 minutes with lukewarm water, take off the mouth tray from the LED light and clean the mouth tray.
- Repeat this process for 7 days every day.

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EN Warnings

- For use only on natural teeth.
- Apply whitening gel only on teeth. Avoid gums and lips.
- Do not use if gums are sensitive, damaged or inflamed.
- Not suitable for children younger than 12 years.
- Do not use if allergic to any ingredients. If irritation, swelling or sensitive teeth symptoms occur, consult a dental professional or doctor.
- If a significant amount is swallowed, please call a doctor. More than 4 applications per day are not recommended.
- Do not use when pregnant or breastfeeding.
- May stain or discolour fabric.

Dispose
Dispose of the article and the packaging materials in accordance with current local regulations.

The crossed out wheeled bin symbol means that this product shall not be disposed of with normal household waste. Electronic and Electrical Equipment not included in the selective sorting process are potentially dangerous for the environment and human health due to the presence of hazardous substances. Please dispose of responsibly at an approved waste or recycling facility.

Used batteries may not be disposed of in the household waste. Consumers are legally obligated to dispose of batteries at a collection point in their community/city district or at a retail store. The purpose of this obligation is to ensure that batteries are disposed of in an environmentally congruent manner. Only dispose of batteries when they are completely drained.

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    Battery Li-MnO2Button Cell CR2032 3.0V (included)

Overschiepstraat 63
1062 XD Amsterdam
The Netherlands
info@optismile.eu
www.optismile.eu

3 x 3ml e
art. no.: 749
batch no.: 2240253

03/2025

ARTWORK SPECIFICATION

FILE INFORMATION

Name: Teeth Whitening Kit - MANUAL
Article NO.: 749
Batch NO.: 2240253
PO NO.:
Date: 10.08.2022

PRINT SPECIFICATION

Size: 40 mm x 200 mm x 170 mm
Pre print base: Opaque White 
No

CMYK  
Pantone colors used:

PACKAGING SPECIFICATION

Type of packaging:
Colorbox

UV + Emboss
Foil effect

COATING:
Matt 
Matt varnish
Glossy

PAPER SPECIFICATION

Type of Paper:
Regular cardboard
***Using corrugated cardboard is NOT ALLOWED**
Thickness:

COMMENTS / NOTES

When Pantone® colours are used, the colours must be matched with the official Pantone® guide.

DO NOT alter this artwork without discussing with the design team. If the design causes printing problems, discuss this with Karsten International team.

This artwork is protected by copyright and may **NOT** be shared with other parties without explicit permission of Karsten International.

CONTACT

DESIGN:
Tais De langhe t.delanghe@karsten.nl

PRODUCT MANAGMENT:
Ada a.xie@karsten.nl

Karsten International B.V.
Overschiepstraat 63, 1062 XD
Amsterdam, The Netherlands Phone: +31 (0)20 6325676