# &me.

## INSTRUCTION MANUAL EAR CLEANER

11025065

## INTRODUKTION

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Vi tager forbehold for trykfejl.

## INTRODUCTION

Thank you for choosing a product from &me. You have acquired a quality product which meets all applicable performance and safety standards in the EU. In order to ensure proper treatment and a long product life, we recommend that you comply with the instructions below.

Please read the manual and especially the safety instructions carefully before using the product. Keep this manual and provide it for referral along with the product when others are using it.

## GENERAL SAFETY INSTRUCTIONS

- Improper use of the product can cause personal injury and damage.
- Only use the product for what it is intended. The manufacturer is not liable for damages arising from improper use or handling.
- Always place the product on a dry, flat and stable surface well away from table edge or similar.
- The product is intended for household use only. The product is not for professional use.
- Never touch the product, cord or plug with wet or damp hands.
- Do not expose the product to direct sunlight, high temperatures, humidity, dust or corrosive substances.
- Never leave the product when it is turned on.
- When the product is in use, it should be kept under constant supervision. Children should always be supervised when using the product to ensure that they do not play with the product. The product is not a toy.
- The product may have sharp parts and must, therefore, be kept out of reach of children.
- Only use the accessories that come with the product or that are recommended by the manufacturer.
- The product must not be used by people with reduced sensitivity,

physical or mental disability, or people who are not able to operate the product unless they are supervised or instructed by a person responsible for their safety.

## **CORD AND PLUG**

- Do not let the cord hang over the edge.
- Keep the cord and the product away from heat sources, hot objects and open flames.
- Unplug the power cord before cleaning and if the product is not in use. Do not pull the cord when removing the plug from the socket, rather grasp the plug itself.
- Make sure that the cord is fully extended.
- Check regularly that the cord and the plug are not damaged and do not use the product if this is the case, or if it has been dropped on the floor, dropped in water or damaged in another way.
- If the product or plug is damaged, the product must be inspected and, if necessary, repaired by an authorised repairer. Never try to repair the product yourself. Please, contact the store to see if there is a repair that falls under warranty. Tampering of the product will void the warranty.

## **RULES OF COMPLAINTS THE WARRANTY DOES NOT APPLY:**

- If the above is not observed.
- If there has been unauthorised intervention of the product.
- If the product has been mishandled, subjected to rough treatment or some other form of damage.

## **PRODUCT DISPOSAL**

Used product should not be disposed as household waste. According to legislation, the product should be disposed of properly so that the materials can be recycled in an environmentally friendly manner. Further information is available from the responsible local authority or local disposal company. The product

is equipped with the following logo:


1


Scan the QR code  
Download APP

2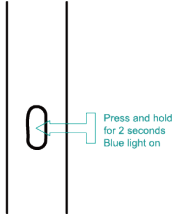

Press and hold the device's power button to turn it on.

3

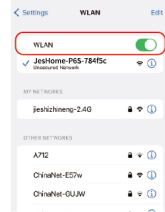
Enable Wi-Fi in your mobile settings.

4

Tap "Not connected."  
The device will automatically connect to the product's Wi-Fi.

5

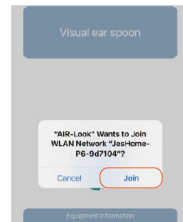Connect to the product's Wi-Fi  
A pop-up will appear for Wi-Fi:  
JesHome-P6-XXX.  
Tap "Join."

6

Tap "Start" to begin using the device.

7

The device is now ready for use.

## **PRODUCT PARAMETERS**

Ambient Temperature: -10°C to 50°C  
Battery Capacity: 130mAh  
Charging Time: 40 minutes  
Working Time: 40 minutes  
Network Standard: IEEE 802.11 b/g/n (2.4Ghz)  
Image Sensor: CMOS  
Lens Diameter: 3.5mm  
Image Transmission Resolution: 8MP  
Working Voltage: 3.7V-4.2V  
Input Voltage: 5V/1A  
Product Size: 14mm × H145mm  
Product Weight: 14.2g  
Charging Interface: Type-C

## **ACCESSORIES**

- Type-C charging cable × 1
- Silicone earpick sleeves × 5

## **MOBILE PHONE SYSTEM SUPPORT**

1. For Android phones: Supports version 5.0 and above.
2. For iPhones: Supports iOS 10.0 and above.

## **CAUTIONS**

1. It is recommended to carefully clean the lens with a cotton swab.
2. Before use, check your surroundings. Do not use it near moving people or animals, in a vehicle, or in unstable conditions. Keep your body steady.
3. This product is not suitable for children under 3 years old.
4. To avoid accidental injury, children must not use this product alone.
5. Do not expose the device to a humid environment for an extended period or immerse it in liquid.
6. This product has a built-in polymer battery, which must be charged at least once a month if not used frequently.
7. Use a 5V charger to charge the device. Do not use the device while it is charging.
8. Avoid exposing the product to direct sunlight.
9. Do not place heavy objects on the device.
10. During use, the product may heat up slightly (up to approximately 35 °C), which is normal.
11. Do not submerge the product in water for cleaning, as this may cause damage.

## TROUBLESHOOTING

S/N	Fault	Solution
1	No response after startup	Charge the device using the provided cable. Charging time is 40 minutes.
2	Camera indicator does not light up	Turn off the device, then restart or reconnect to WiFi.
3	APP crashes	Uninstall the app, scan the QR code, and install the latest version.
4	Silicone sleeve is loose	Remove the silicone sleeve, reinsert the earpick slowly, or replace the sleeve.
5	Blue indicator flashes quickly	If the battery is low, charge the device using the provided cable.
6	Unstable connection; frequent disconnections or freezing during use	Restart the earpick and reconnect to the app.
7	Image does not display properly after connecting the product to the app	Disconnect your phone from the earpick's WiFi, restart the earpick, and reconnect.

If the above steps do not resolve the issue, please contact the place of purchase.

Component Name	Restricted substances and its chemical symbols					
	(Pb)	(Hg)	(Cb)	(Cr(VI))	(PBB)	(PBDE)
Circuit board	○	○	○	○	○	○
Casing	○	○	○	○	○	○
Wires	○	○	○	○	○	○
Plastic silicone accessories	○	○	○	○	○	○

Note: "○" means the percentage of this hazardous substance does not exceed the standard limit.

### TECHNICAL DATA

Article No. 11025065

### PRODUCER

Epiq ApS

Krøyer Kielbergs Vej 3, 3. th

8660 Skanderborg, Denmark

[www.epiq.dk](http://www.epiq.dk)

We cannot be held responsible for any printing errors.